

OC-PRO-003 WI-006 Revision 6	Instructions For Use	
	For Fresh & Go Aligner and Retainer Cleaning Tablets	

Instructions for Use (IFU) Fresh & Go Aligner and Retainer Cleaning Tablets

REF	DEVICE DESCRIPTION (TRADE NAMES)
9001-237	Appliance Cleaning tablet, pack of 36
9001-238	Appliance Cleaning tablet, pack of 96
9001-239S	Appliance Cleaning tablet, pack of 4
9001-239PK2	Appliance Cleaning tablet, pack of 2

Product Description and Intended Use:

This is a product intended for cleaning a medical device (an orthodontic appliance) but not for disinfecting or sterilising. The claimed action is to help maintain the clarity of the appliance, keep the appliance fresh and help remove stains. The tablet has a cool mint flavour. The users would be wearers of dental/orthodontic appliances, therefore this demographic would be pre-adolescents, adolescents and adults.

Issue No.: 6	Page 1 of 5	Date of Issue: 16-Sep-2026
--------------	-------------	----------------------------

OC-PRO-003 WI-006	Instructions For Use	
Revision 6	For Fresh & Go Aligner and Retainer Cleaning Tablets	

Instructions for Use:

- 1



Dissolve tablet in 280ml of warm but **NOT** hot water.
- 2



Add appliance to effervescent solution for 15–30 mins, or 15 mins exactly for wire appliances.
- 3



Wash hands thoroughly.
- 4



Rinse in water at room temperature for 30 seconds.
- 5



Discard the solution after one use.

Issue No.: 6	Page 2 of 5	Date of Issue: 16-Sep-2026
--------------	-------------	----------------------------

OC-PRO-003 WI-006	Instructions For Use	
Revision 6	For Fresh & Go Aligner and Retainer Cleaning Tablets	

Indications for Use:

Suitable for cleaning the following removable dental and orthodontic appliances:

- Clear Retainers
- Wire Retainers
- Bruxing Appliances
- Night Guards
- Clear Aligners
- Sports Mouthguards
- TMJ Appliances

Storage and Disposal:

Store in a cool and dry place. Discard solution after one use and dispose of tablets and solution in accordance with local regulations.

Performance Characteristics and Clinical Benefit:

The clinical benefit of the subject device is demonstrated by its performance in cleaning the dental/orthodontic appliance.

Warnings:

Do not use if you are sensitive to any of the ingredients. Keep out of reach of children. Do not swallow/place in mouth, the tablet or the solution. Harmful if inhaled. Avoid contact with eyes. If solution/tablet makes contact with eyes rinse thoroughly with water. Avoid excessive skin contact. Immediately seek medical advice if severe irritation develops and/or tablet or solution is ingested. Discard cleaning solution after one use. Dispose of tablets and solution in accordance with local regulations. Do not mix with any other cleaners.

Contraindications:

This product is contraindicated for any individuals with allergies to the listed ingredients. Not to be used on any appliances other than those indicated for use. Not to be used directly on the appliance without diluting it in the specified amount of water.

Reporting Serious Incidents:

Any serious incident that has occurred in relation to use of this product should be reported without undue delay to Ortho-Care and the competent authority for the country in which the incident occurred.

Issue No.: 6	Page 3 of 5	Date of Issue: 16-Sep-2026
--------------	-------------	----------------------------

OC-PRO-003 WI-006 Revision 6	Instructions For Use	
	For Fresh & Go Aligner and Retainer Cleaning Tablets	

Ingredients:

Sodium Bicarbonate, Citric Acid Anhydrous Granular, Potassium Peroxymonosulfate, Sodium Carbonate Anhydrous, Sodium Percarbonate, Sodium Laurylsulfate, Tetra Acetyl Ethylenediamine (TAED), Sodium Benzoate, VP/VA Copolymer, Polyethylene Glycol-180 (PEG-180), Peppermint Oil, Indigo Dye.

Symbols that Apply to All Fresh & Go Aligner & Retainer Cleaning Tablets

Symbol:	Symbol Meaning and Information that accompanies the symbol:	Symbol:	Symbol Meaning and Information that accompanies the symbol:
	Medical Device		Lot/Batch Number See product package for lot number.
	Unique Device Identifier See product package for UDI number.		Catalogue Number See product packaging for catalogue number.
	Use by date		Do not re-use
	Keep Dry		Keep away from sunlight
	Do Not Use if Package is Damaged		Consult Instructions for Use
	CE Mark		UKCA Mark

Issue No.: 6	Page 4 of 5	Date of Issue: 16-Sep-2026
--------------	-------------	----------------------------

OC-PRO-003 WI-006 Revision 6	Instructions For Use For Fresh & Go Aligner and Retainer Cleaning Tablets		
Symbol:	Symbol Meaning and Information that accompanies the symbol:	Symbol:	Symbol Meaning and Information that accompanies the symbol:
	Manufacturer Ortho-Care (UK) Ltd, 1 Riverside Estate, Saltaire, BD17 7DR, West Yorkshire UK. Tel: +44 (0)1274 533233 Email:info@orthocare.co.uk		EU Authorised Representative Advena Limited, Tower Business Centre, 2 nd Flr, Tower Street, SWATAR BKR 4013 MALTA
	H317: May cause an allergic skin reaction H319: Causes serious eye irritation H335: May cause respiratory irritation		Caution

Imported into: Austria (AT), Czechia (CZ), Denmark (DK), Estonia (EE), Finland (FI), Germany (DE), Hungary (HU), Italy (IT), Latvia (LV), Lithuania (LT), Norway (NO), Slovenia (SI), Sweden (SE) and Switzerland (CH),

By Lifco Dental AB, Verkmästaregatan 1,
745 39 Enköping,
Sweden

medicaldevices@lifcodental.se

Imported into Poland by ORTOTECH,
Czerniakowska 131/15, 00-720 Warszawa, Poland,
info@ortotech.pl

Issue No.: 6	Page 5 of 5	Date of Issue: 16-Sep-2026
--------------	-------------	----------------------------